



12 MONTH SLIP RESISTANT STEALTH® UNDER CUSHION CHAIR PAD model: 70980

- Antimicrobial
- Latex Free
- Antibacterial

Initial power up: Press the button on the small box attached to the cord for 2 seconds. If moved to a new chair or if chair cushion is moved, press the button for 2 seconds to recalibrate the pad.

▲WARNING▲

INSTRUCTIONS FOR USE

1. Before first use, write the date on the sensor pad with an indelible marker.
2. Check this sensor pad. Remove from service and advise the vendor if there is any damage.
3. Center the sensor pad across the width of the chair, under the chair cushion in the area where the patient's buttock will rest.

CAUTION: To avoid possible skin breakdown bare skin on legs should not be placed directly on the slip resistant surface. The pad should be placed entirely under a cushion or covered by a suitable soft material between the pad and skin.

PAD PLACEMENT

This pad is designed to work under the chair cushion if placed on a flat stationary surface of the chair frame as illustrated. There are several advantages of an under-chair cushion placement where the pad is:
(1) Away from bodily fluids (2) does not restrict access to the patient (3) is less likely to be damaged (4) does not impede the pressure reducing properties of the chair cushion.

TEST FOR EACH PATIENT BEFORE EACH USE.

TEST FOR SLIP RESISTANCE BEFORE EACH USE BY PLACING MODERATE HAND PRESSURE OVER THE SURFACES WHILE ATTEMPTING TO MOVE THE PRESSURE POINT FORWARD.

SEALED SENSOR PAD CARE



This sealed sensor pad will resist water and bodily liquids only as long as the sealed cover remains entirely intact. Discontinue use of this sensor pad if worn or torn. The pad may be disinfected with a disinfectant suitable for vinyl surfaces.

▲WARNING▲

FAILURE TO FOLLOW THESE INSTRUCTIONS COULD RESULT IN SERIOUS INJURY

Use of this sensor pad: (1) is not appropriate for all individuals or chair cushions (2) not a substitute for visual monitoring by a caregiver and (3) will not prevent falls. This sensor functions only with approved monitors. Test carefully before each use.

Read and follow these instructions fully before each use.

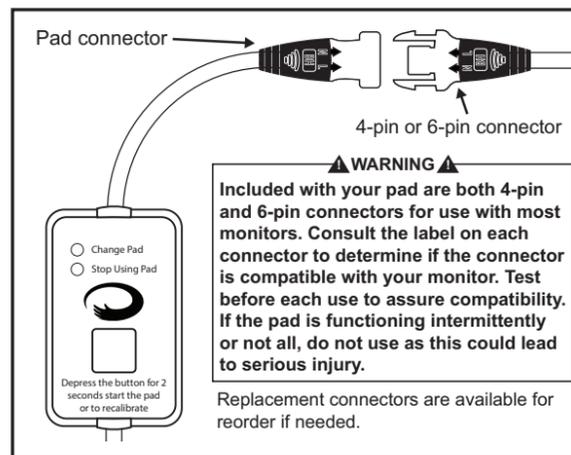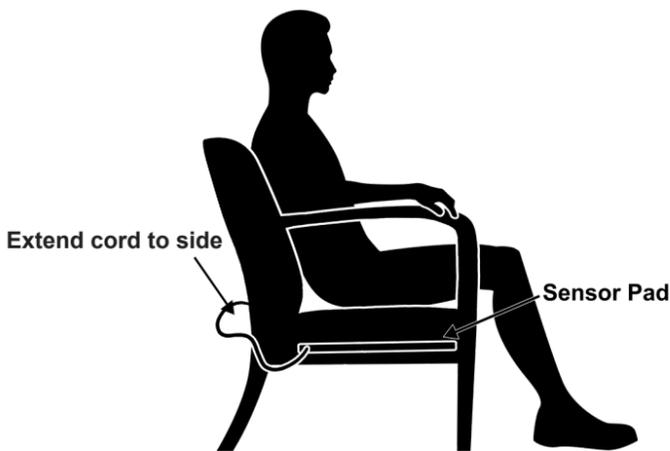

18 19 20 21 A B C D E F G H I J K L

DATE PUT IN USE:
____/____/____

DATE TO DISCONTINUE USE:
____/____/____

IMPORTANT: There is a raised area in the corner where the cord enters the pad. Do not position patient on this corner area.



THIS SIDE UP Store Flat. Do Not Fold or Crease.

▲WARNING▲

This pad should be discarded and replaced after its specified service time frame has elapsed. Failure to do so could result in malfunction or injury.

This pad is designed to be placed below the chair cushion. Place the pad on a flat, solid and stationary surface under the chair cushion; otherwise the pad can be bent and may not function correctly.

Test the system with each new patient before use. Failure to follow these instructions could result in injury or malfunction.

TESTING YOUR SYSTEM

Before each use, test your system by pressing down on the chair cushion over the sensor pad for at least three (3) seconds and release. The alarm should sound within 2 seconds of when you release pressure. Repeat the test at least twice (2) using different locations over the sensor pad. If there is a failure, discontinue use.

LIMITED WARRANTY

This sensor pad is warranted against defects of workmanship or manufacture for a period of 30 days and on a prorated basis or until the pad wears out whichever is earlier. The sole remedy will be repair or replacement if returned to the vendor. There is no other remedy or warranty, express or implicit, including merchantability or fitness. Manufacturer shall not be liable for any loss of use of sensor pad or the other incidental or consequential losses, expenses or damages.

This sensor pad is designed to be installed by the user. It is the responsibility of the user to see that it is properly installed and tested with each use.

Failure to test this sensor before each use, misuse, or inappropriate cleaning will void the limited warranty.

PAD REPLACEMENT

This Stealth® pad must be replaced at the end of its service life, which is 12 months. Wear and tear may reduce service life. With three months remaining of the 12 month service life: light blinks once every half second. With one month remaining: light blinks rapidly three times every half second. If pad has expired the light becomes solid.

You should make sure a replacement unit will be available at the end of service.

If you have questions, call or visit:

UK Freephone:
0800-038-3513

US Phone:
+1 402- 476-2858

www.turun.co.uk

www.safepresence.com

© 2018. Made in China.
ISO Certified Manufacturing
DOC 21.7080.11
patent pending

